
## 3M™ Steri-Strip™

First Aid Skin Closures GB

### Application Instructions

1. Ensure that bleeding has stopped and any dirt and grit has been removed from the cut. Clean the skin for at least 5 cm around the cut and then dry thoroughly.
2. Peel open the plastic pouch, remove skin closures and tear-off card end tab. To reduce curling, the strips should be removed at a 45 degree angle to the card.
3. At the mid-point of the wound apply one half of the first skin closure strip to the skin and press firmly into place. Exactly push the wound edges together. Press the free half of the closure strip firmly into place, taking care not to apply under tension.
4. Complete wound closure with additional strips spaced approximately 2-3 mm apart to allow for drainage of exudate.
5. To remove Steri-Strip first aid skin closures apply light pressure at the centre of the skin closure, then alternately peel back each end towards the cut site, and gently remove the whole strip.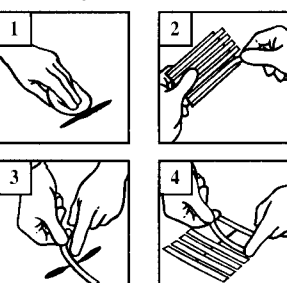

STORE IN COOL, DRY PLACE.  
STERILE UNLESS ENVELOPE  
DAMAGED OR OPEN.

**CE 0086**  
Made in USA by  
**3M Health Care**  
St. Paul, MN 55144-1000, USA  
**3M Health Care**  
D-41453 Neuss, Germany  
3M and Steri-Strips are trademarks of the  
3M Company  
YP-2763-0074-3 / 1205

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**3M Nederland B.V.**
**3M Svenska AB**
**3M Suomi Oy**